


Instructions «MagniLink Vision BASIC»

1. Panel

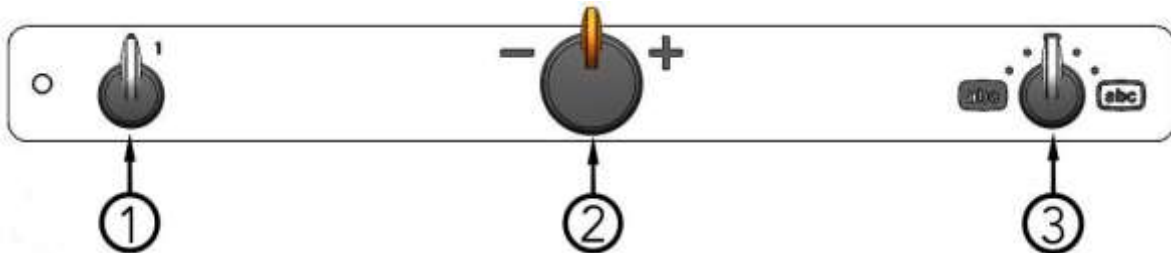

1.1. Power on/off

Turn this knob to switch the unit on or off.

1.2. Magnification control


Turn this knob to the left to decrease the magnification or turn it to the right to increase the magnification. With this knob the magnification can be set to 16 different levels.

1.3. Colour picture mode and high contrast mode

With the knob in the upright starting position the colour picture, with natural colours, is displayed on the screen. Turn the knob to the left for a negative high contrast mode and to the right for a positive high contrast mode. There are four combinations in the standard configuration:

| Background/Text | Background/Text |
|-----------------|-----------------|
| White/Black | Yellow/Black |
| Black/White | Black/Yellow |

2. Reading table

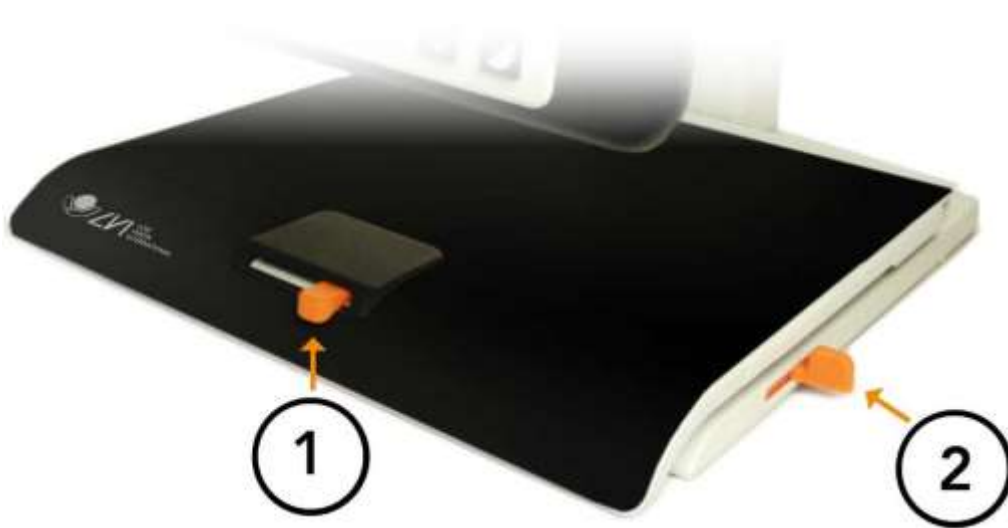

2.1. Side way direction friction adjustment

Set the side way friction by moving this handle. With the brake in the left end position, the reading table is unlocked. To increase friction, push the side way brake to the right.

2.2. Depth way direction friction adjustment

Set the depth way friction by moving this handle. With the brake in the near end position, the reading table is unlocked. To increase friction, push the depth way brake towards the back.